









Standard Worksheet

Process: Cleaning Glucose Meter ('Nova Stat Strip') and Supply Container Tote
Desired Goal or Outcome: To prevent cross contamination

1	2	3	4	5	6	7	8
							
<p>Perform hand hygiene (HH).</p> <p>Put on gloves as per the PCRA and remove a facility approved disinfectant wipe (e.g., Accel Intervention)</p>	<p>Squeeze out extra fluid so wipe is damp and NOT dripping.</p>	<p>If visibly soiled, remove soiling with wipe, discard and take another wipe.</p> <p>Clean/disinfect the meter front; ensure contact with screen and buttons.</p> <p>Allow required wet contact time with screen and buttons.</p> <p>Allow required wet contact time to kill organisms (Accel Intervention 1 minute).</p>	<p>Clean and disinfect both sides of meter ensuring good contact with the grip.</p> <p>Allow required wet contact time.</p>	<p>Clean and disinfect the bottom of meter.</p> <p>Don't forget the feet! Allow required wet contact time.</p>	<p>Clean and disinfect the top.</p> <p>When cleaning the insertion port, fold the wipe to create a corner.</p> <p>Clean with a swiping motion, similar to cleaning out the corner of an eye.</p> <p>Allow required wet contact time. Place meter on an unused wipe.</p>	<p>If using Supply Container:</p> <p>Wipe all surfaces of the container; place container on a wipe.</p> <p>Allow required wet contact time. Only place clean equipment in the container.</p>	<p>Remove gloves and perform hand hygiene.</p> <p>Pick up the cleaned items and discard wipes.</p> <p>Return meter to storage.</p>