

Process: **Cleaning Table, Keyboard, Mouse, Wrist Support and Push Handles on Work Station on Wheels (WOWs)**



- | | | | | | | | |
|--|---|--|--|---|---|---|---|
| <p>1. Perform hand hygiene (HH). Put on gloves if indicated by the PCRA and remove a facility approved disinfectant wipe (e.g., Accel Intervention®)</p> | <p>2. Squeeze out extra fluid so wipe is damp and NOT dripping.</p> | <p>3. If visibly soiled, remove soiling with wipe and discard. Take another wipe, and clean and disinfect the table in a circular motion, starting in the center and working outwards, scrubbing, ensuring good contact with surfaces. Allow appropriate wet contact time (1 minute for Accel Intervention) to provide sufficient kill of organisms.</p> | <p>4. With the same wipe, clean and disinfect the push handle.</p> | <p>5. Clean and disinfect around the edge, as well as the underside of the keyboard. Allow required wet contact time.</p> | <p>6. Start at one end and clean up and down “vertical” or back and forth “horizontal” while moving the wipe across the keyboard. Pass the wipe over the keys 3 times. Allow required wet contact time.</p> | <p>7. Clean and disinfect the mouse (including the left and right buttons and dial). Allow required wet contact time.</p> | <p>8. Clean and disinfect the wrist support. Allow required wet contact time. Discard wipes. Remove gloves. Perform HH.</p> |
|--|---|--|--|---|---|---|---|

Clean WOWs:

1. After use for EACH patient/resident/client.
2. Prior to removal from an Isolation Room.