









## Standard Worksheet

**Process:** Cleaning Table, Keyboard, Mouse, Wrist Support and Push Handles on Work Station on Wheels (WOWs)

**\*AFTER** use for EACH patient/resident/client and **\*PRIOR** to removal from an Isolation Room

1	2	3	4	5	6	7	8
							
<p>Perform hand hygiene (HH).</p> <p>Put on gloves as per the PCRA and remove a facility approved disinfectant wipe (e.g., Accel Intervention)</p>	<p>Squeeze out extra fluid so wipe is damp and NOT dripping.</p>	<p>If visibly soiled, remove soiling with wipe, discard.</p> <p>Take another wipe, and clean and disinfect the table in a circular motion, starting in the center and working outwards, scrubbing, ensuring good contact with surfaces.</p> <p>Allow appropriate wet contact time (1 minute for Accel Intervention) to provide sufficient kill of organisms.</p>	<p>With the same wipe, clean and disinfect the push handle</p>	<p>Clean and disinfect around the edge, as well as the underside of the keyboard.</p> <p>Allow required wet contact time.</p>	<p>Start at one end and clean up and down “vertical” or back and forth “horizontal” while moving the wipe across the keyboard.</p> <p>Pass the wipe over the keys 3 times.</p> <p>Allow required wet contact time.</p>	<p>Clean and disinfect the mouse (including the left and right buttons and dial).</p> <p>Allow required wet contact time.</p>	<p>Clean and disinfect the wrist support.</p> <p>Allow required wet contact time.</p> <p>Discard wipes.</p> <p>Remove gloves.</p> <p>Perform HH.</p>