

Standard Worksheet

Process: *HoverMatt® Cleaning and Disinfection*

Desired Goal or Outcome: *To prevent infection transmission and to clean and disinfect HoverMatt®, at minimum, after every patient use*

There are **3 different types of HoverMatt®** available:

1. Reusable Heat-Sealed
2. Reusable Double-Coated
3. Disposable

***NOTE** Ensure staff are aware of the type of HoverMatt® available on site as each may have different cleaning, disinfection, laundering and drying instructions based on manufacturer's recommendations.

HOVERMATT® CLEANING AND DISINFECTION: Patient Room Process	2-4
HOVERMATT® CLEANING AND DISINFECTION: Laundry Process	5
HOVERMATT® CLEANING AND DISINFECTION: Patients on Additional Precautions	6-9
HOVERMATT® CLEANING AND DISINFECTION: Operating Room and/or Labour & Delivery	10
HOVERMATT® PREVENTATIVE MAINTENANCE	11

STAFF AND EQUIPMENT FOR CLEANING/DISINFECTION

- ❖ Minimum of 2 healthcare workers (HCWs) to complete this work.
- ❖ Personal Protective Equipment (PPE) as required based on HCW's Point of Care Risk Assessment (PCRA) and/or patient's ADDITIONAL PRECAUTIONS. PPE may include: gowns, gloves, procedure mask, eye protection
- ❖ Facility approved disinfectant wipes
- ❖ Facility approved disinfectant Ready to Use (RTU) liquid
- ❖ Clean linen cover (e.g., reusable linen sheet)

GENERAL CONSIDERATIONS: HOVERMATT®

- ❖ During care of patient use *HoverMatt® Disposable Protector Sheets®* OR other reusable linen covers (e.g., sheets) to protect HoverMatt® from gross soiling
 - ❖ In Operating Room (OR) and Labour & Delivery (L&D) settings, use fluid proof, *HoverMatt® Disposable Protector Sheets®* to prevent gross soiling of HoverMatt®
- ❖ Use impervious sleeve to cover accordion air hose to prevent transmission risk

CLEANING AND DISINFECTION: HOVERMATT®

- ❖ Clean/disinfect HoverMatt® on the stretcher/HoverJack®/bed/other;
- DO NOT CLEAN/DISINFECT HOVERMATT® ON THE FLOOR**
- ❖ **Inflate** HoverMatt® during cleaning/disinfection
 - ❖ Clean/disinfect all surfaces (top, bottom, sides, pull handles, belt straps and clips), air supply inlet/unit, and storage cart
 - ❖ Keep wet for manufacturer's required contact time (e.g., Accel Intervention requires 1 minute wet contact time)
 - ❖ Allow to completely dry before reuse or packaging equipment (air holes and liquid from disinfectant or laundry process poses risk for microorganism growth)

Standard Worksheet

REUSABLE HOVERMATT® CLEANING AND DISINFECTION: Patient Room Process

STEP #	DESCRIPTION	KEY POINTS / IMAGES	WHO
1	<p>Laterally transfer patient then:</p> <ul style="list-style-type: none"> <input type="checkbox"/> Deflate HoverMatt® on patient bed <input type="checkbox"/> Remove HoverMatt® from under patient <input type="checkbox"/> Move HoverMatt® onto stretcher/HoverJack® bed/other to clean/disinfect 	<p>Clean/disinfect HoverMatt® before it leaves the patient environment</p>	<p>Facility designated staff (e.g., persons involved in transfer, Housekeeping staff).</p>
2	<ul style="list-style-type: none"> <input type="checkbox"/> Perform hand hygiene (HH). <input type="checkbox"/> Put on gloves if HoverMatt® is visibly soiled. <input type="checkbox"/> Reinflate HoverMatt® to clean/disinfect. <input type="checkbox"/> Remove <i>HoverMatt® Disposable Protector Sheet®</i>, dispose in garbage in patient environment OR remove other reusable linen cover and place into soiled laundry bin. <input type="checkbox"/> Check top and bottom surfaces of HoverMatt® for visible soiling. If visibly soiled clean area with disinfectant wipes then discard wipes. 	<p>Clean/disinfect HoverMatt® even if protective cover used, even if not visibly soiled.</p> <p>Don gloves, additional PPE as required.</p>	
3	<ul style="list-style-type: none"> <input type="checkbox"/> Take off gloves and gown (if worn). <input type="checkbox"/> Perform HH. Put on new gloves and eye protection. <input type="checkbox"/> Saturate top surface of HoverMatt® with RTU liquid. <input type="checkbox"/> Spread liquid with disinfectant wipes to cover all top surfaces, sides and straps of HoverMatt®. Discard wipes. <input type="checkbox"/> Keep wet for manufacturer's required contact time. <input type="checkbox"/> Wipe off excess disinfectant if pooled (e.g., crevices). <input type="checkbox"/> Allow top surface of HoverMatt® to dry completely. 	<p>RTU liquid must be used with disinfectant wipes; wipes alone do not have enough solution to saturate HoverMatt®.</p> <p>DO NOT use rags; they absorb RTU liquid so the HoverMatt® does not stay wet long enough to be disinfected.</p> <p>If splashing of RTU liquid is anticipated, put on PPE (e.g., eye protection).</p> <p>Clean/disinfect HoverMatt® from head-toe.</p> <p>Accel Intervention wipes require a 1 minute wet contact time.</p> <p>HoverMatt® must remain inflated while drying. Top surface of inflated HoverMatt® takes approximately 5-10 minutes to dry.</p>	


Standard Worksheet

REUSABLE HOVERMATT® CLEANING AND DISINFECTION: Patient Room Process

STEP #	DESCRIPTION	KEY POINTS / IMAGES	WHO
4	<p>1st HCW: <input type="checkbox"/> Tilt/lift HoverMatt® off stretcher/HoverJack®/bed/other</p>	<p>Require a clean surface to place the <i>cleaned/disinfected HoverMatt® top side</i>, while the rest of the HoverMatt® is cleaned/disinfected.</p> <p>HH is required before handling clean linen cover to prevent contamination.</p>	
	<p>2nd HCW: <input type="checkbox"/> Remove protective cover (if used) and place into soiled laundry bin (reusable linen cover) or discard into waste bin (HoverMatt® Disposable Protector Sheet).</p>		
	<p><input type="checkbox"/> Take off gloves, perform HH and place a clean linen cover on the stretcher/HoverJack®/bed/other.</p>		
5	<p><input type="checkbox"/> Flip cleaned/disinfected and dry top surface of HoverMatt® onto clean linen cover of stretcher/HoverJack®/bed/other.</p>	<p>Do not place cleaned/disinfected areas of HoverMatt® onto soiled surface. Reposition air supply unit (if needed) when HoverMatt® is flipped over.</p>	
6	<p>Both HCWs: <input type="checkbox"/> Remove gloves <input type="checkbox"/> Perform HH <input type="checkbox"/> Put on new gloves</p>	<p>Clean/disinfect exposed underside surface of HoverMatt® from head to toe.</p>	<p>Facility designated staff (e.g., persons involved in transfer, Housekeeping staff).</p>
	<p><input type="checkbox"/> Saturate underside surface of HoverMatt® with RTU liquid. <input type="checkbox"/> Spread liquid with disinfectant wipes to cover all underside surfaces, sides, outer edges, air intake connector, flaps, handles/buckles and HoverMatt® straps. <input type="checkbox"/> Discard wipes <input type="checkbox"/> Keep wet for manufacturer's required contact time. <input type="checkbox"/> Wipe off pooled excess disinfectant (e.g., crevices).</p>	<p>PPE required when cleaning underside surface due to air holes which may spray germs into eyes and mouth.</p> <p>Prevent cleaned/disinfected parts of HoverMatt® (e.g., straps, flaps) from touching contaminated surfaces</p>	
7	<p>Keep HoverMatt® inflated while drying. Cleaned/disinfected HoverMatt® must be dry before deflating, folding and placing back into storage cart.</p>	<p>Air supplying the mattress speeds drying. Underside surface of HoverMatt® dries in ~5 minutes. A moist, wet mattress facilitates microbial growth.</p>	

Standard Worksheet

REUSABLE HOVERMATT® CLEANING AND DISINFECTION: Patient Room Process

STEP #	DESCRIPTION	KEY POINTS / IMAGES	WHO
8	While waiting for underside to dry clean/disinfect Air Supply Unit/Storage Cart.	Ensure wipe is not saturated to avoid damaging air supply unit motor. Do not squeeze disinfectant liquid directly on air supply unit. 	
	1st HCW: <input type="checkbox"/> hold the hose		
	2nd HCW: <input type="checkbox"/> Clean/disinfect exposed surfaces of air supply unit and storage cart with disinfectant wipes		
	Both HCWs <input type="checkbox"/> Wait for HoverMatt® to dry completely before proceeding to next step.		
9	<input type="checkbox"/> turn off air supply unit; HoverMatt® will start to deflate. <input type="checkbox"/> Disconnect hose from HoverMatt®. <input type="checkbox"/> Clean/disinfect HoverMatt® connection site with disinfectant wipes	Allow cord to dry before folding it.	Facility designated staff (e.g., persons involved in transfer, Housekeeping staff)
	2nd HCW:		
	1st HCW: <input type="checkbox"/> Remove/dispose of hose cover, hold end of hose to prevent falling onto contaminated surface	Return HoverMatt®/storage cart to designated storage location.	
	2nd HCW:		
	<input type="checkbox"/> Wipe entire hose and open end of hose (that plugs into HoverMatt®) with disinfectant wipes. <input type="checkbox"/> Place cleaned/disinfected hose in storage cart. <input type="checkbox"/> Remove gloves. Perform HH.		
	1st HCW:	Storage cart must be stored in a clean and dry environment.	
	<input type="checkbox"/> Clean/disinfect cord (from unit end to plug end) with disinfectant wipes. <input type="checkbox"/> Place cord into cleaned/disinfected storage cart. <input type="checkbox"/> Remove gloves. Perform HH.		
Both HCWs			
	<input type="checkbox"/> Complete folding HoverMatt. <input type="checkbox"/> Place in storage cart. <input type="checkbox"/> Exit patient environment. Perform HH.		
	Launder HoverMatt® if still soiled after initial clean/disinfection. See REUSABLE HOVERMATT® LAUNDERING PROCESS		

Standard Worksheet

REUSABLE HOVERMATT® CLEANING AND DISINFECTION: Laundry Process

STEP #	DESCRIPTION	KEY POINTS / IMAGES	WHO
1	<p>REUSEABLE HoverMatt® must be laundered:</p> <ul style="list-style-type: none"> ❖ If still soiled after initial cleaning/disinfection ❖ If used for a patient on Contact, Droplet/Contact, Airborne/Contact or Containment Precautions ❖ If used with a patient in the OR or L&D Room <p>DO NOT place soiled HoverMatt® on HoverMatt® storage cart when sending to Laundry.</p> <p>Refer to Site Specific HoverMatt® Laundering Instructions.</p> <div style="background-color: #e0e0e0; padding: 10px; margin-top: 20px;"> <p>IMPORTANT:</p> <p>Throw out DISPOSABLE HoverMatt® after patient no longer requires it. Do not machine wash or dry as it cannot tolerate the process.</p> </div>	<p>Create a site specific Laundering Procedure that includes:</p> <ul style="list-style-type: none"> ❖ Linen bag to transport soiled HoverMatt® ❖ Provision of space to launder ❖ Appropriate washing machine with recommended temperature for cleaning/disinfection -160F or 71C. ❖ Ability to completely dry HoverMatt® (space to hang to dry, use air supply unit, or dryer with specific temperature), and ❖ Number of HCWs to move it when wet. <p>Frequency of HoverMatt® usage may be impacted due to time needed for laundering/drying.</p> <p>Follow manufacturer's laundering instructions: http://www.hovermatt.com/hovermatt.html</p> <div style="background-color: #e0e0e0; padding: 10px; margin-top: 20px;"> <p>IMPORTANT:</p> <p>Do not put the DOUBLE-COATED HoverMatt® in the dryer; allow to air dry completely.</p> </div>	<p>Facility designated staff (e.g., persons involved in transfer, Housekeeping staff).</p>

Standard Worksheet

HOVERMATT® CLEANING AND DISINFECTION: Patients on Additional Precautions

STEP #	DESCRIPTION	KEY POINTS / IMAGES	WHO
1.0	<p>Patients on Contact Precautions (including Airborne/Contact, Contact, Droplet/Contact) OR Containment Precautions use:</p> <p>DISPOSABLE single patient use HoverMatt®</p> <p style="text-align: center;">OR</p> <p>REUSABLE HoverMatt® may be used but <u>MUST BE LAUNDERED</u> after patient no longer requires.</p> <p>Put on PPE as per Additional Precautions and PCRA (gloves, gown, eye protection, mask/N95 respirator as appropriate).</p>	<p>After patient no longer requires HoverMatt® (e.g., discharge, isolation discontinued):</p> <p>a. <u>DISPOSABLE</u> HoverMatt® must be discarded (<u>cannot be laundered or reprocessed</u>)</p> <p>b. <u>REUSABLE</u> HoverMatt® <u>must be</u> laundered (remove gross soiling before laundering; See steps 1.1 – 1.9 below)</p> <p>DISPOSABLE HoverMatt® can be dedicated and reused on same patient:</p> <ul style="list-style-type: none"> ❖ Clean/disinfect if grossly soiled during use ❖ Dispose if still visibly soiled after cleaned/disinfected ❖ Obtain new Disposable HoverMatt® as required <p>Do not take storage cart into patient environment if it can be avoided. If storage cart has been in patient environment, clean/disinfect it before placing cleaned/disinfected HoverMatt® into cart and before leaving patient environment.</p>	<p>Facility designated staff (e.g., persons involved in transfer, housekeeping).</p>
1.1	<p>If Disposable or Reusable HoverMatt® is grossly soiled, clean/disinfect area.</p> <ul style="list-style-type: none"> ❖ Deflate and remove HoverMatt® from under patient ❖ Move HoverMatt® onto stretcher/HoverJack®/bed/other 	<p>Clean/disinfect grossly soiled HoverMatt® before it leaves patient environment</p>	<p>Facility designated staff (e.g., persons involved in transfer, housekeeping).</p>


Standard Worksheet

HOVERMATT® CLEANING AND DISINFECTION: Patients on Additional Precautions

STEP #	DESCRIPTION	KEY POINTS / IMAGES	WHO
1.2	<ul style="list-style-type: none"> <input type="checkbox"/> Re-inflate HoverMatt® to clean/disinfect <input type="checkbox"/> Remove HoverMatt® <i>Disposable Protector Sheet</i> and discard into garbage in patient environment OR remove other reusable linen cover and place into soiled laundry bin. 	Clean/disinfect HoverMatt® even if protective cover used.	Facility designated staff (e.g., persons involved in transfer, housekeeping).
1.3	<ul style="list-style-type: none"> <input type="checkbox"/> Check top surface of HoverMatt® for visible soiling. If visibly soiled: <ul style="list-style-type: none"> ❖ Clean area with disinfectant wipes ❖ Discard wipes <input type="checkbox"/> Saturate soiled area of HoverMatt® with RTU liquid. <input type="checkbox"/> Spread liquid with disinfectant wipes to cover soiled surface. Discard wipes. <input type="checkbox"/> Keep wet for manufacturer's required contact time (e.g., Accel Intervention needs 1min. wet contact time). Do not rinse. <input type="checkbox"/> Wipe off excess disinfectant if pooled (e.g., crevices) with disinfectant wipe. <input type="checkbox"/> Ensure HoverMatt® is dry before deflating or placing back onto patient bed. 	<p>RTU liquid must be used in conjunction with disinfectant wipes; wipes alone do not contain enough solution to saturate HoverMatt®.</p> <p>Do not use rags to clean/disinfect HoverMatt®; they absorb RTU liquid so the HoverMatt® will not stay wet long enough to be disinfected.</p> <p>Clean/disinfect HoverMatt® from head to toe depending on soiling.</p> <p>HoverMatt® <u>must remain inflated while drying.</u></p> <p>Top surface of inflated HoverMatt® takes ~ 5 minutes to dry.</p>	
1.4	<p>Check HoverMatt® underside surface for visible soiling; if soiled, clean/disinfect area</p> <hr style="border-top: 1px dashed black;"/> <p>1st HCW: <input type="checkbox"/> Tilt/lift HoverMatt® off stretcher/HoverJack®/bed/other</p> <hr style="border-top: 1px dashed black;"/> <p>2nd HCW: <input type="checkbox"/> Remove gloves <input type="checkbox"/> Perform HH and put on gloves <input type="checkbox"/> Place clean linen cover on stretcher/HoverJack®/bed/other</p> <hr style="border-top: 1px dashed black;"/> <p>1st HCW: <input type="checkbox"/> Flip topside of HoverMatt® to expose soiled underside surface</p>	<p>HH required before handling clean linen cover to prevent contamination. Clean linen cover provides clean area for top surface of HoverMatt® to rest while completing cleaning/disinfection of the underside of the HoverMatt®.</p> <p>Air supply unit may need repositioning when HoverMatt® flipped over.</p>	Facility designated staff (e.g., persons involved in transfer, Housekeeping staff).

Standard Worksheet

HOVERMATT® CLEANING AND DISINFECTION: Patients on Additional Precautions

STEP #	DESCRIPTION	KEY POINTS / IMAGES	WHO
1.5	<p>Both HCWs</p> <ul style="list-style-type: none"> <input type="checkbox"/> Remove gloves <input type="checkbox"/> Perform HH and <input type="checkbox"/> Put on new set of gloves <hr/> <ul style="list-style-type: none"> <input type="checkbox"/> Clean soiled area with disinfectant wipes <input type="checkbox"/> Discard wipes. <input type="checkbox"/> Saturate soiled area of HoverMatt® with RTU liquid. <input type="checkbox"/> Spread liquid with disinfectant wipes to cover soiled surface. Discard wipes. <input type="checkbox"/> Keep wet for manufacturer's required contact time. <input type="checkbox"/> Wipe off excess disinfectant if pooled (e.g., crevices). <input type="checkbox"/> Keep HoverMatt® inflated while drying. <input type="checkbox"/> Ensure HoverMatt® is dry before placing back on patient bed. <input type="checkbox"/> If HCWs leave patient environment while HoverMatt® is drying, remove PPE and apply clean PPE upon re-entry. 	<p>Air holes on underside of HoverMatt® may spray germs when RTU liquid applied. Put on PPE (e.g., eye protection such as safety glasses, procedure mask) to prevent splashing into eyes and mouth.</p> <p>RTU liquid must be used in conjunction with disinfectant wipes; wipes alone do not contain enough solution to saturate HoverMatt®.</p> <p>Do not use rags to clean/disinfect HoverMatt®; they absorb RTU liquid so the HoverMatt® will not stay wet long enough to be disinfected.</p> <p>Clean/disinfect HoverMatt® from head-toe depending on soiling.</p> <p>Air supplying the mattress speeds up drying. Underside surface of inflated HoverMatt® takes ~ 5 minutes to dry. A moist, wet mattress facilitates microbial growth and material breakdown.</p>	
	1.6	<p>Upon discharge or discontinuation of Additional Precautions</p> <ul style="list-style-type: none"> <input type="checkbox"/> Discard DISPOSABLE HoverMatt® <input type="checkbox"/> Place REUSABLE HoverMatt® into designated laundry bag per site laundering instructions. 	<p>Air hose, air supply unit, and storage cart must be cleaned and disinfected before removing from patient environment.</p>
1.7	<p>Cleaning/disinfection of Air Supply Unit/hose/cart</p> <hr/> <p>1st HCW:</p> <ul style="list-style-type: none"> <input type="checkbox"/> Hold the hose <hr/> <p>2nd HCW:</p> <ul style="list-style-type: none"> <input type="checkbox"/> Clean/disinfect exposed surfaces with disinfectant wipes <hr/> <p>HCWs must wait for HoverMatt® to be completely dry before moving to next step.</p>	<p>Ensure wipe is not saturated to avoid damaging the air supply unit motor. Do not squeeze disinfectant liquid directly on the air supply unit.</p>	

Standard Worksheet

HOVERMATT® – CLEANING AND DISINFECTION: Patients on Additional Precautions

STEP #	DESCRIPTION	KEY POINTS / IMAGES	WHO
1.8	<p>1st HCW:</p> <ul style="list-style-type: none"> <input type="checkbox"/> Hold connection site of hose <input type="checkbox"/> Remove cover starting from air supply unit. <input type="checkbox"/> Hold end of hose to prevent contamination 	Allow cord to dry before folding it.	
	<p>2nd HCW:</p> <ul style="list-style-type: none"> <input type="checkbox"/> Clean/disinfect hose & connector site with disinfectant wipes. 		
	<p>1st HCW:</p> <ul style="list-style-type: none"> <input type="checkbox"/> Remove gloves, perform HH, put on new gloves, take new wipes, and hold hose. <input type="checkbox"/> Clean hose connector exterior and interior. <input type="checkbox"/> Place clean/disinfected hose into cart. <input type="checkbox"/> Clean/disinfect electric cord (from unit to plug end) with disinfectant wipes. 		
1.9	<p>1st HCW:</p> <ul style="list-style-type: none"> <input type="checkbox"/> Place electric cord into cleaned/disinfected storage cart. <input type="checkbox"/> Push cleaned/disinfected storage cart to doorway. 	Return storage cart to designated storage location.	Facility designated staff (e.g., persons involved in transfer, Housekeeping staff).
	<p>Both HCWs</p> <ul style="list-style-type: none"> <input type="checkbox"/> Remove PPE <input type="checkbox"/> Perform HH <input type="checkbox"/> Remove cart from patient room <p>If cart outside patient environment, do not bring inside. Clean all components that were in patient room prior to removing. If cart had been taken into patient room, clean/disinfect air supply unit, hose and electric cord in patient room (see above) before removing and placing back on cart.</p>	Storage cart must be stored in a clean and dry environment.	
2.0	<p>Patients on <u>Airborne or Droplet Precautions (including Airborne/Contact or Droplet/Contact)</u></p> <p>Always replace air filter after use for airborne, airborne/Contact Precautions.</p> <p>Contact department responsible for preventative maintenance of HoverMatt.</p> <p>Once patient no longer requires REUSABLE HoverMatt®: clean/disinfect HoverMatt® before removal from patient room</p>	<p>A REUSABLE, dedicated HoverMatt® may be used for patients on Airborne or Droplet precautions, but must be cleaned/disinfected after each use.</p> <p>Follow the REUSABLE HoverMatt® Cleaning and Disinfection (Patient Room Process) procedure and/or Laundering procedure.</p>	

Standard Worksheet

HOVERMATT® CLEANING AND DISINFECTION: Operating Room and/or Labour & Delivery

STEP #	DESCRIPTION	KEY POINTS / IMAGES	WHO
--------	-------------	---------------------	-----

<p>1.0</p>	<p>Use</p> <p>a. DISPOSABLE single patient-use HoverMatt®</p> <p style="text-align: center;">OR</p> <p>b. REUSABLE HoverMatt®, but <u>MUST LAUNDER</u> after use.</p> <p>Use <i>HoverMatt Disposable Protector Sheet</i>® in L&D and/or OR settings.</p>	<p>HoverMatt® usage in L&D and/or OR areas increases risk of cross contamination from blood and body fluids.</p> <p>Before HoverMatt® usage, determine which type of HoverMatt® is appropriate (DISPOSABLE versus REUSABLE) based on anticipated level of soiling. See OESH Safe Work Procedure.</p> <p><u>DISPOSABLE HoverMatt®:</u></p> <ul style="list-style-type: none"> ❖ Can be dedicated and reused on the same patient ❖ Clean/disinfect it grossly soiled during use ❖ Throw out if still visibly soiled after cleaned/disinfected ❖ Obtain a new Disposable HoverMatt® as required ❖ Discard once patient no longer requires it ❖ Cannot be laundered or reprocessed ❖ Dispose of according to level of soil (e.g., biohazardous vs. general waste) <p><u>REUSABLE HoverMatt®:</u></p> <ul style="list-style-type: none"> ❖ Must be laundered if used in L&D and/or OR settings ❖ Before laundering, if grossly soiled, clean/disinfect REUSEABLE HoverMatt® as per <i>Reusable HoverMatt® Cleaning and Disinfection procedure section</i>. <p>If patient is on Additional Precautions, see <i>HoverMatt® Cleaning and Disinfection for Patients on Additional Precautions</i> procedure section.</p> <p>For Laundering direction, see <i>Reusable HoverMatt®: Laundry Process</i>.</p>	<p>Facility designated staff (e.g., persons involved in transfer, Housekeeping staff).</p>
------------	--	--	--

Standard Worksheet

HOVERMATT® CLEANING AND DISINFECTION: Preventative Maintenance

STEP #	DESCRIPTION	KEY POINTS / IMAGES	WHO
--------	-------------	---------------------	-----

<p>1.0</p>	<p>Air Filter Cleaning:</p> <p>Per User Manual instructions. Clean air filter monthly, or according to facility's preventative maintenance schedule.</p> <p>Remove Filter from the air supply and hold it under warm running water.</p> <p><u>Filter must air dry completely</u> before placed back into air supply unit.</p> <p>Air Filter Replacement:</p> <p>Replace filter annually at a minimum, or when it:</p> <ul style="list-style-type: none"> ❖ Is clogged with debris that does not break free when it is washed ❖ Begins to lose its shape or deteriorates ❖ Remains discolored or "dirty" after rinsing ❖ No longer stays in place on the motor 	<p>Always replace filters after use for airborne precautions.</p>	<p>Site staff responsible for Air Filter Maintenance</p>
<p>2.0</p>	<p>HoverMatt® Maintenance:</p> <p>Before use, perform a visual inspection on the HoverMatt® to ensure.</p> <ul style="list-style-type: none"> ❖ There is no visible damage that makes it unusable ❖ The HoverMatt® has all of its patient safety straps and handles ❖ There are no tears or holes that prevent it from inflating. 	<p>Refer to manual for all appropriate parts.</p> <p>If any damage is found that causes the system not to function as intended, remove from use and return to HoverTech.</p>	