

## Standard Work Sheet

**Process:**    **Reusable C-MAC® IMAGER with Disposable Blade Reprocessing**

**Desired Goal or Outcome: To prevent cross contamination by cleaning/disinfection of this item after each use**

Step#	Description	Key Points / Images	Who
Immediately after each use:			
1	Discard the C-MAC® S blade	Blade is single use	Nurse/RRT
2	Check imager for visible soiling		Nurse/RRT
3	<b>If C-MAC® imager is NOT visibly soiled:</b>		
3.1	Perform hand hygiene		Nurse/RRT
3.2	Gather cleaning supplies: Accel Intervention Wipes and lint free cloth		Nurse/RRT
3.3	Perform hand hygiene		Nurse/RRT
3.4	Put on gloves		Nurse/RRT
3.5	Place imager on one side of a clean, disinfected surface	Clean/disinfect surface the Imager will be placed on Place 2-3 disinfectant wipes on other side of table or surface	Nurse/RRT
3.6	Wipe screen with 70% isopropyl alcohol and let dry Wipe remaining surfaces with Accel Intervention Wipes Use 3 wiping motions with firm pressure for each surface Keep surfaces wet for 1-minute contact time As items are cleaned and disinfected, place on opposite side of the table	Use enough wipes to wet entire surface  Repeat wiping if needed to keep surfaces wet	Nurse/RRT
3.7	Put wipes in the garbage Take off gloves and put in garbage Perform hand hygiene		Nurse/RRT
3.8	Allow imager to air dry	Do not wipe with dry cloth or towel	Nurse/RRT
3.9	Return imager to its designated location or store in a designated clean storage area		Nurse/RRT
4	<b>If C-MAC® imager is visibly soiled:</b>		
4.1	Wipe the entire exterior of the imager with a soft, lint-free cloth moistened with water or accel intervention wipes	This is pre-clean of equipment and is necessary for it to be adequately cleaned and disinfected	Nurse/RRT
4.2	Send imager to MDR for reprocessing	Place imager in an MDR container for protection	Nurse/RRT
4.3	Perform hand hygiene		Nurse/RRT

