

Standard Worksheet

Process: *Cleaning and Disinfection of Kego™ Oxygen Tank Bags*

Preamble: *Kego™ oxygen tank bags are dedicated to a single patient/resident/client (P/R/C)*

- Desired Goal or Outcome:**
- Prevention of infection transmission in all healthcare settings*
 - Reduce healthcare acquired infections*
 - Standardize cleaning and disinfection for Kego™ oxygen tank bags*
 - Provide clean oxygen tank bags prior to use*

CLEANING FREQUENCY	<i>Between users, when visibly soiled, after additional precautions discontinued and when dedicated to one P/R/C (ideally) once a week or at minimum once per month</i>
RESPONSIBILITY	<i>Whoever assists the P/R/C with the oxygen tank is responsible for sending the Kego™ oxygen tank bag for laundering</i>

STEP	PREPARATION
1	Perform hand hygiene and put on Personal Protective Equipment (PPE) as indicated by Additional Precautions and/or Point of Care Risk Assessment (PCRA). PPE may include: <ul style="list-style-type: none"> • Gloves • Gown, eye protection and mask based on PCRA - if splash is anticipated
2	Inspect bag for breakage, or wear – if damaged, flag for repair or dispose (follow manager’s or equipment team’s direction for disposal)

STEP	LAUNDERING OF KEGO™ OXYGEN TANK BAGS
1	After use, if the bag is not visibly soiled, store in a clean dry place in the P/R/C room/bed space
2	Put bag in laundry for any of the following reasons: <ul style="list-style-type: none"> • visibly soiled • when it is to be used on a different P/R/C • it is the scheduled laundry time • after discontinuation of additional precautions not mentioned below
3	Kego™ oxygen tank bags used with a P/R/C on additional precautions for bed bugs or Crusted/Norwegian Scabies are to be disposed of as bags will not tolerate high temperature laundering (<i>follow manager's or equipment team's direction for disposal</i>)
4	Kego™ oxygen tank bags used with a P/R/C on additional precautions for typical Scabies are to be placed into an air and water-tight plastic bag for 7 days and then laundered
5	Kego™ oxygen tank bags used with a P/R/C on additional precautions for Lice (Pediculosis) are to be placed into an air and water-tight plastic bag for 10 days and then laundered
6	Remove gloves if used; remove other PPE if used
7	Perform hand hygiene
8	Laundering Instructions: Kego™ oxygen tank bags are to be machine washed using a cool temperature cycle and hung to dry