

Patient Scales

STATUS: ☐ Passed ☐ Service Required

Site:	Mechanical <u>or</u> Electrical (circle one)	
Date:	Inspected by:	Location:
Manufacturer:	Model:	Serial Number:

Task	Criteria	Test Weight(s) Used	Weight Displayed	Pass	Fail
Zero Calibration	Display reads zero when scale is on/no weight on platform	N/A		<input type="checkbox"/>	<input type="checkbox"/>
Weight Accuracy (Example – For test weight of 50 lbs, scale should read within +/- 1% or between 49.5 - 50.5 lbs)	Scale reads weight that is within +/- 1% of test weight(s) used			<input type="checkbox"/>	<input type="checkbox"/>

PREVENTATIVE MAINTENANCE	
Done	
<input type="checkbox"/>	Clean exterior and accessories
<input type="checkbox"/>	Lubricate (if applicable)
<input type="checkbox"/>	Calibrate

NOTES

Note: All completed Patient Scale templates are to be maintained at the site to provide a record of calibration and maintenance that has occurred