

## ZOLL AED PLUS CHECKLIST:

Are the following team reminders in place?

YES/NO/COMMENTS:

1.	<p>Do you have a reminder in place of when to review the Zoll AED Plus to test and inspect to ensure the Zoll AED is ready for use?</p> <ul style="list-style-type: none"> <li>To check the Expiry Dates of the AED Pads for both Adult and Child?</li> <li>To check batteries?</li> <li>To test the AED is properly working?</li> </ul> <p>NOTE: For example, at end of month with medication inventories</p>	
2.	<p>Do you have a reminder in place of when to review the Zoll AED Plus video for practice?</p>	

Is the equipment ready for use and properly labelled?

3.	<p>Ensure the "on off" is labeled Note: label comes with the unit and available from the manufacture</p>	
4.	<p>Ensure "Lift to Open" is labeled Note: label comes with the unit and available from Zoll Customer service</p>	
5.	<p>AED pads expiration date is labeled and secured onto the AED with the sticky backing Note: label comes with the unit and with the replacement pads call Zoll Customer service</p>	
6.	<p>Do you have a battery replacement reminder label next to the On/Off button? Note: comes with the unit and available from Zoll Customer service</p> <p>Refer to Zoll AED Plus Video and Operating Instructions for battery replacement instructions NOTE: "For AED Plus units running software version 5.32 or higher, replace batteries every 5 years or if unit prompts. For earlier software versions, replace batteries every 3 years."</p>	
8.	<p>Do you have the Adult pads connected? NOTE: Zoll AED recommends pre-connect the Adult pads not the Pediatric pads.</p>	
9.	<p>Have you checked the Expiry Dates of the AED Pads for both Adult and Child?</p>	
11.	<p>Do you have the supplies that came with the AED kit?</p> <ul style="list-style-type: none"> <li>Gloves</li> <li>Scissors</li> <li>Razor</li> <li>Face Shield</li> <li>Clean Wipe</li> </ul>	
12.	<p>Is the Zoll AED plus Operators Guide with the machine? If not, refer to the manufacturer site</p>	

October 6, 2016